


Procedure Card

Section I : Sample Collection


- 1 Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.

- 2 Remove Sample Collector Protector.

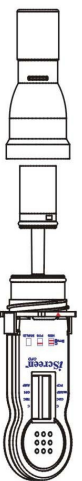


- 3 Insert Sponge end of collector into mouth.

- Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure **complete saturation**. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing Sponge.
- Hand the saturated collector over to the tester.



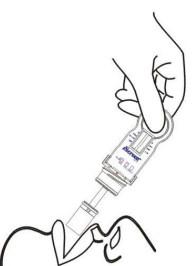


At least
3 minutes

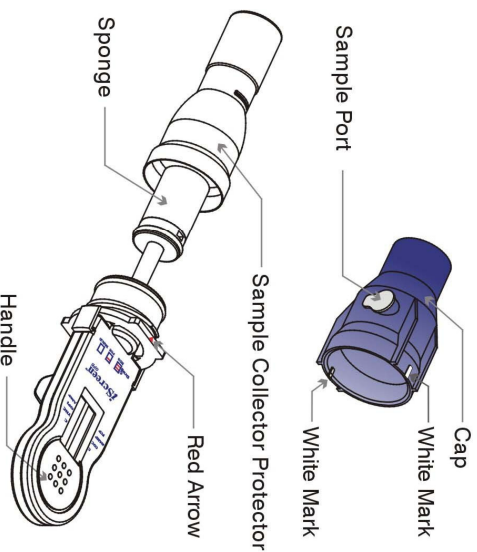


Section II : Testing

- 1 A. Align the Red Arrow on the device with either one of the White Marks on the Cap. Insert the collector vertically into the Cap and **press down firmly until the Cap reaches the thread.**
 B. Twist the Handle clockwise 180° to tighten the Cap until the Red Arrow lines up with the other White Mark.
 C. Place the test device horizontally on a clean and level surface with the result window facing up.

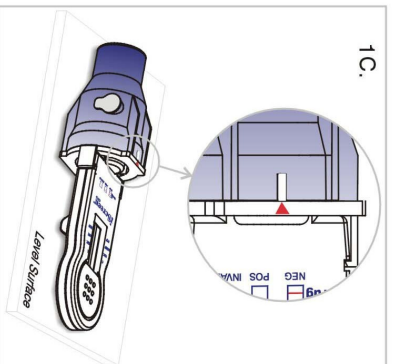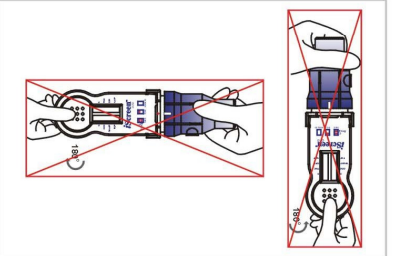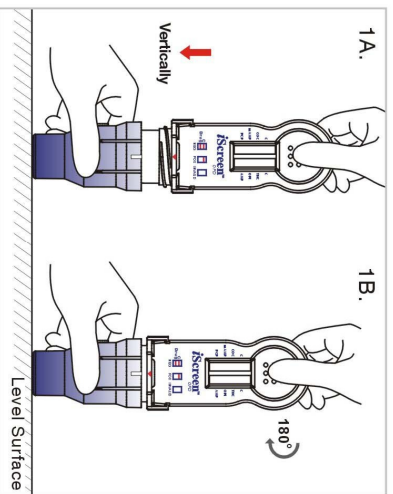

- 2 Read results at 10 minutes. Do not read results after 1 hour.

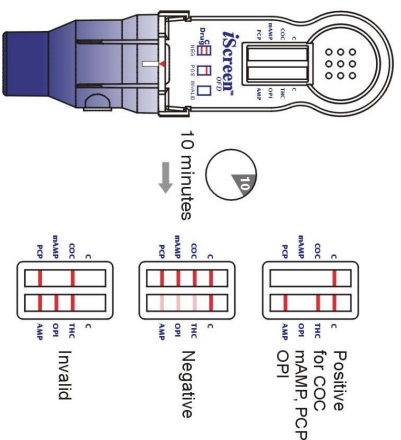

- All positive results are presumptive and should be confirmed by an alternate method. The preferred confirmatory method is GC/MS or GC/MS/MS.

- 3 If results are positive, secure Cap with Security Seal and send to laboratory for confirmation.

